



# No more jam jars. Just right-first-time urine specimens.

## Peezy: gold standard urology diagnostics

**Clinically proven Peezy Midstream collects a pure urine specimen for right-first-time analysis, diagnosis and treatment. It also delivers huge NHS cost and efficiency savings by reducing repeat tests and patient appointments.**

Setting a new standard in urology diagnostics, Peezy Midstream virtually eliminates heavy mixed growth and contamination providing the most accurate dipstick and laboratory results ever. A new option to the standard Peezy Midstream system is the **Peezy Primary 10ml kit** that removes the need for decanting in the lab, introducing yet another layer of cost savings and efficiency to a common and essential procedure. Peezy Midstream is accurate, quick, clean and kind.

## Goodbye

- Repeat testing and patient visits
- Contamination and heavy mixed growth
- Unnecessary antibiotic prescribing
- Soiled containers

## Hello

- Accurate dipstick and lab testing
- Cost and efficiency savings
- Infection control excellence
- Improved patient care

## Each kit contains:

**PE40/41** Peezy, genital wipe, 30ml universal container

**PE50/51** Peezy, genital wipe, 10ml primary container

### Sales and distribution

[sales@forte-medical.com](mailto:sales@forte-medical.com)

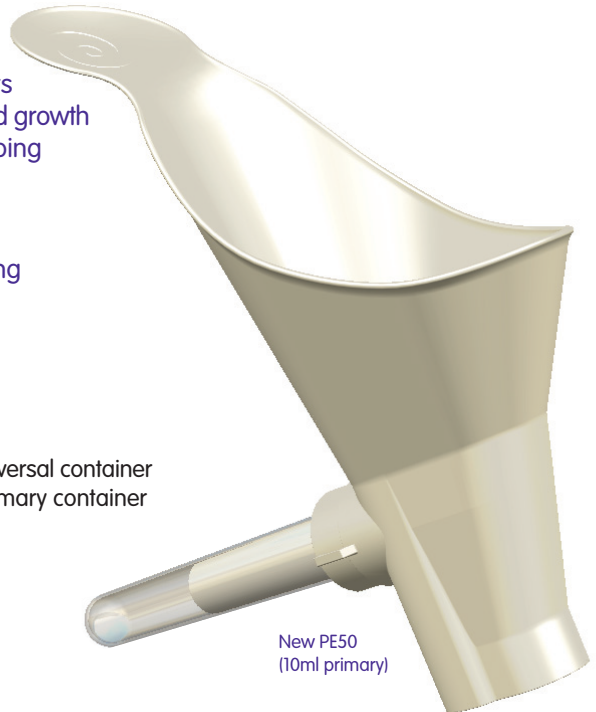
### Product codes

PE40 (30ml standard)

PE41 (30ml boric)

PE50 (10ml standard)

PE51 (10ml boric)



New PE50  
(10ml primary)

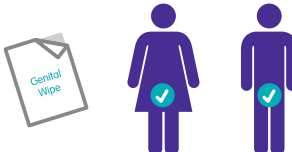
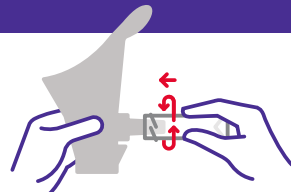


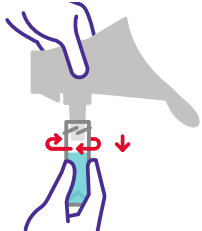
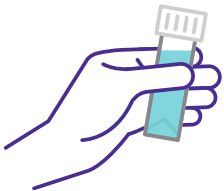


“Peezy is a welcome innovation; spillage is minimised and toilet hygiene is maintained for general infection prevention. The improvement in possible unreliable results is impressive, suggesting a significant financial saving due to the common nature of the investigation into LUTS assessments.”

**Mr Wai Man Chow**, Consultant Urological Surgeon, Pennine Acute Hospitals NHS Trust

**How to use Peezy in 5 easy steps - read before use**

Peezy works best with a full bladder

<p><b>1 Use genital wipe</b></p> 	<p><b>2 Fit bottle</b></p> <ul style="list-style-type: none"> <li>Remove lid.</li> <li>Screw bottle on to Peezy.</li> </ul> 
<p><b>3 Pee</b></p> <p><b>Sit well back</b></p> 	<ul style="list-style-type: none"> <li>Hold Peezy as shown and pee.</li> <li>Pee also runs into the toilet.</li> <li><b>When finished count to 10</b> - this lets Peezy drain.</li> </ul>  <p><b>Hold over toilet or urinal</b></p>
<p><b>4 Remove bottle</b></p> <ul style="list-style-type: none"> <li>Hold bottle upright.</li> <li>Unscrew bottle.</li> <li>Replace lid.</li> </ul>  <p><b>Put Peezy in bin provided</b></p>	<p><b>5 Give bottle to nurse</b></p> 



**Owned and patent protected:**  
 Forte Medical Limited, London EC2A 4SD  
[forte-medical.com](http://forte-medical.com)



**Manufactured in the UK:**  
 Boddingtons Plastics Limited, Kent TN12 9QU  
[boddingtons.co.uk](http://boddingtons.co.uk)



Issue 4 March 2017